


Measuring Guide for K9 HipLign™




Make sure your pet is standing squarely. If they are weak in the rear, you can use a towel to support their hind end.



LENGTH: Measure along the back, from the shoulder blades to base of tail.

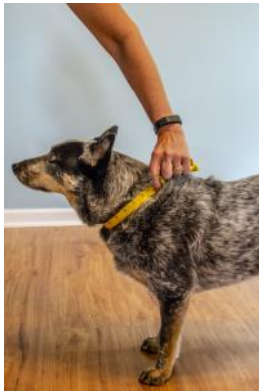

NECK: Measure your pet's neck at the base, just in front of the shoulder blade.

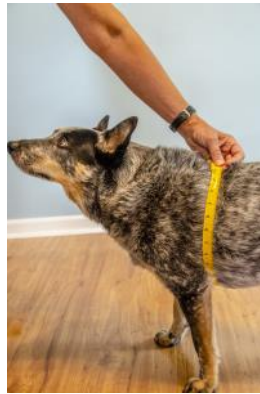

CHEST: Measure around the chest, just behind the front legs.



THIGH: Measure around the thigh, at the widest part.

K9MOTION

K9MOTION.COM



@K9Motion



@K9Motion



K9_Motion

Enter these measurements on your order form, and you are set! If you have any questions about the sizing or your measurements, please send photos of you measuring your pet (with the tape measure visible for us to read) in the positions shown above to us at :

Contact@K9Motion.com